



FEATHER SURGICAL BLADE

New Product

No.350 / No.391

No.350

The blade shape allows to perform a micro incision.


Width: 2.0mm

Width: 2.0mm

No.391

Allows to perform a safer incision and high operability.

Total length is 3mm longer and blade length is approximately a half of No.390.

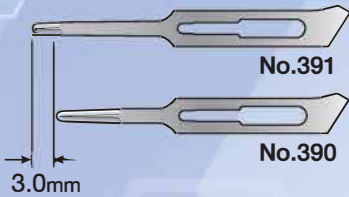

Feather surgical blades No.350/No.391 are the small sized surgical blades that can be handled in a narrow space of the oral cavity. These blades enable to perform the micro incision especially for the microsurgery. Please use No.350/No.391 as well as other Feather surgical blades No.370, No.390, and No.390C for the micro surgery according to the cases.



The blades can attach to Feather Surgical Handle.

Applicable handles



Distributed by:
Medix21 Surgical
29 Jarden Mile Ngauranga 6035
p 0800 633 4921 / 021 734 762
e camille@medix21.co.nz
w medix21surgical.co.nz

MADE IN JAPAN

Example use of Feather Surgical Blade No.350/No.391

No.350




It allows the delicate incision in the thin gingiva and creates an accurate flap design.




A small sized surgical blade is reachable between the teeth during the periodontal regeneration surgery; allows an oblique incision in the interdental space and creates the accurate flap.



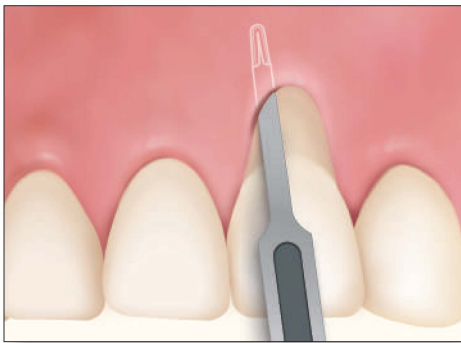


No.391

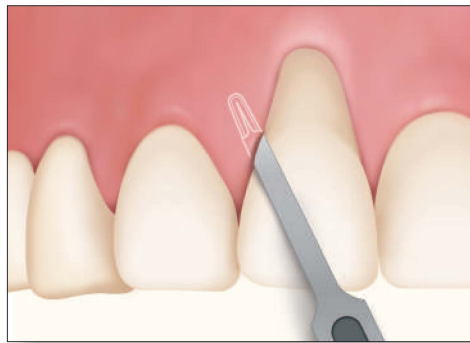

Total length is longer than No.390 and it allows a deeper envelope flap.



A shorter blade than No.390 allows a safe and accurate gingiva incision during the envelope flap surgery.

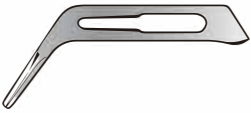	Blade shape	Blade thickness	Cutting edge	Package
New	No.350 	0.38mm	Sharp end on both sides	10 blades per box
New	No.391 		Sharp end on one side	
	No.370 		Sharp end on both sides	
	No.390 		Sharp end on one side	
	No.390C 		Sharp end on both sides	



FEATHER SAFETY RAZOR CO., LTD.
OVERSEAS TRADE DIVISION

3-70, OHYODO MINAMI 3-CHOME, KITA-KU, OSAKA 531-0075, JAPAN
PHONE: +81-6-6458-1638 FAX: +81-6-6458-1611
URL www.feather.co.jp/ E-mail overseas@feather.co.jp



9001:メテカナル刃
13485:メテカナル